




PROFESSIONAL AND  
INNOVATIVE SOLUTIONS  
FOR NEGATIVE PRESSURE  
WOUND TREATMENT



[www.confortnpwt.com](http://www.confortnpwt.com)

## NEGATIVE PRESSURE WOUND THERAPY

is an active wound closure system that affect the healing factors in acute and chronic wounds and play an active role in shortening the healing time. *Indications for use* ; Orthopedic Trauma, Soft Tissue Trauma, Skin Grafts, Pressure Ulcers, Leg Ulcers, Diabetic foot, Burns, Open Abdomen Wounds, Management of Surgical Infections and Other Surgical Wounds.

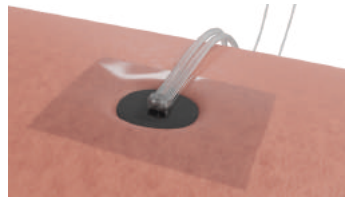

The vacuum generated by the negative pressure unit is sent to the wound through the system elements. According to this principle, negative pressure (vacuum) is applied to the wound surface continuously with different pressure values and time intervals.



**Confort**  
NPWT

PROVIDES 3 DIFFERENT  
NPWT UNITS (PUMPS)

### FEATURES



**C300**



**C300 TS**



**C500**  
elis

<b>DIMENSIONS</b>	255 mm x 230 mm x 135 mm (with canister holder)	255 mm x 230 mm x 135 mm (with canister holder)	170 mm x 110 mm x 80 mm (140mm with canister)
<b>WEIGHT</b>	2.2 kg	2.2 kg	≈ 1 kg
<b>BATTERY TYPE</b>	Li – Ion	Li - Ion	Li - Ion
<b>THERAPY MODES</b>	1. Continuous, 2. Intermittent 3. Instillation	1. Continuous, 2. Intermittent 3. Instillation 4. Intermittent Instillation	1. Continuous, 2. Intermittent, 3. Instillation, 4. Intermittent Instillation
<b>PRESSURE RANGE</b>	Min -25 mmHg , Max -300 mmHg	Min -20 mmHg , Max -250 mmHg	Min -20 mmHg , Max -250 mmHg
<b>BATTERY TIME</b>	9-10 Hours	9-10 Hours	9 - 10 Hours
<b>ALARM</b>	Visual & Audible	Visual & Audible	Visual & Audible



#### Continuous Therapy Mode

Continuous Mode is the most common used treatment mode. It treats the wound by constantly working with the adjusted pressure level.



#### Intermittent Therapy Mode

Intermittent Therapy Mode treats by operating two different adjusted negative pressure values at set time intervals and repeat the transition time cycle according to specified times.



#### Instillation Therapy Mode

Instillation Therapy Mode works by sending the desired solution to the wound, at desired intervals and waiting on the wound surface for the specified time during the negative pressure wound treatment.



#### Intermittent Instillation Therapy Mode

Our new Intermittent Instillation Therapy Mode is combination of intermittent and instillation therapy mode. This Therapy Mode is an effective treatment method that provided by applying two different vacuum values to the wound dressing and by washing the wound.

**C300** ✓  
**C300 TS** ✓  
**C500** ✓

**C300** ✓  
**C300 TS** ✓  
**C500** ✓

**C300** ✓  
**C300 TS** ✓  
**C500** ✓

**C300** X  
**C300 TS** ✓  
**C500** ✓



## PORTABLE WITH INSTILLATION THERAPY MODE

Confort C500 Npwt Therapy Unit is produced with the latest technology, provides 4 different therapy modes and has a touch screen. The 4th and new *Intermittent Instillation Therapy Mode* is, which is the first in the world, with the combination of intermittent and instillation therapy mode. C500 is also the first Npwt device in the World that capable of instillation with its sizes.



## CANISTER 1000ML & 500ML Compatible with C500



The new C500 Canisters contain *carbon filter and bacterial filter* to prevent wound exudate odor. It is unbreakable, unopened and does not drain the liquid inside. Thanks to the liquid *gelling pack* inside, the movement of exudate in the canister is prevented. It is adaptable with all Confort wound closure sets. It is for *single use* only and available in 500ml or 1000ml liquid capacity.

CODE	DESCRIPTION
8680014007503	Confort Npwt Therapy Unit C500
8680014007350	Canister Set-S Stop&Filter 500ml
8680014007336	Canister Set-S Stop&Filter 1000ml

## Confort C300 & C300 TS NPWT

**CONFORT NPWT C300**, provides 3 different therapy modes. Continuous, Intermittent and Instillation therapy modes can be selected by the healthcare professional due to the patients wound treatment requirements. *Confort NPWT C300 Unit* is compatible with standard, silver and Instillation closure sets.



**Confort C300 TS** provides touchscreen experience. Furthermore; provides Intermittent Instillation Therapy mode.

## CANISTER 1000ML Compatible with C300 & C300 TS



CODE	DESCRIPTION
8680014005868	Confort Npwt Therapy Unit C300
8680014007312	Confort Npwt Therapy Unit C300 TS
8680014006735	Canister 1000 ml (Stop & Filter)



# VACUUM ASSISTED WOUND CLOSURE SETS

## Standard Wound Closure Sets

Vacuum Assisted Closure Sets consist of Hydrophobic Medical Foam, Vacuum Pad and Drape. It is compatible with **Confort Npwt C300, C300 TS and C500 Therapy Units**. It is used with Continuous and Intermittent Therapy Modes. Standard closure sets are produced in 3 different sizes according to the sizes of Medical Foams.



CODE	DESCRIPTION	SET CONTENT
8680014004779	Vacuum Assisted Wound Closure Set Small (Hydrophobic)	Medical Foam (1) , Drape (1) , Vacuum Pad (1)
8680014004755	Vacuum Assisted Wound Closure Set Medium (Hydrophobic)	Medical Foam (1) , Drape (2) , Vacuum Pad (1)
8680014004687	Vacuum Assisted Wound Closure Set Large (Hydrophobic)	Medical Foam (1) , Drape (3) , Vacuum Pad (1)

## Wound Closure Sets with Silver

Vacuum Assisted Wound Closure Sets With Silver consist of Medical Foam, Vacuum Pad and Drape. It is compatible with **Confort Npwt C300, C300 TS and C500 Therapy units**. It is used with Continuous and Intermittent Therapy Modes. Silver Sets are recommended to use especially with infected wounds. During the therapy process, the silver medical foam provides continuous silver release. Vacuum Assisted Wound Closure Sets With Silver sets are produced in 3 different sizes according to the sizes of Medical Foams.



CODE	DESCRIPTION	SET CONTENT
8680014004724	Vacuum Assisted Wound Closure Set Small with Silver	Medical Foam (1) , Drape (1) , Vacuum Pad (1)
8680014004717	Vacuum Assisted Wound Closure Set Medium with Silver	Medical Foam (1) , Drape (2) , Vacuum Pad (1)
8680014006957	Vacuum Assisted Wound Closure Set Large with Silver	Medical Foam (1) , Drape (3) , Vacuum Pad (1)


## Wound Closure Sets for Instillation


Vacuum Assisted wound closure sets for instillation Treatment consist of Medical Spiral Form Foam, Vacuum Pad and Drape. Used with Instillation Therapy Mode. Compatible with **Confort Npwt C300, C300 TS and C500 Therapy units**. The spiral foam is specially designed for instillation therapy and provides a homogeneous distribution of the instillation fluid to the wound thanks to its form.





CODE	DESCRIPTION	SET CONTENT
8680014006841	Vacuum Assisted Wound Closure Set Small Spiral For Instillation Treatment	Medical Foam (2), Drape (1,5), Vacuum Pad (1)
8680014006858	Vacuum Assisted Wound Closure Set Medium Spiral For Instillation Treatment	Medical Foam (2), Drape (2), Vacuum Pad (1)
8680014006865	Vacuum Assisted Wound Closure Set Large Spiral For Instillation Treatment	Medical Foam (2), Drape (3), Vacuum Pad (2)



### FEATURES



- 


400-600 Micron Por Range
- 


Soft and Durable Interconnection Hose
- 


Ag  
Special Medical Foam Providing Silver Release for up to 72 Hours
- 


Strong Adhesion In Difficult Areas and Flexibility
- 


Special Package Providing Sterile Application Easiness
- 


Thanks to the spiral form, the instillation fluid is homogeneously applied to the wound.

# VACUUM ASSISTED WOUND CLOSURE SETS

## Abdominal Closure Set

Vacuum Assisted Abdominal Closure Sets consist of Hydrophobic Medical Foam, Vacuum Pad, Organ Protector Film and Drape. It is compatible with Confort Npwt C300, C300 TS and C500 Therapy units. It is used with Continuous and Intermittent Therapy Modes.



### FEATURES

-  Perforated 400-600 Micron Medical Foam Suitable for Abdominal Application
-  Organ Protective Barrier Film
-  Skin-Friendly Silicone Foam

CODE	DESCRIPTION	SET CONTENT
8680014006933	Vacuum Assisted Abdomen Management System	Medical Foam (2) , Drape (4) , Vacuum Pad (1), Organ Protector Film

## Incision Sets

Incision Sets are applied in the treatment of incision lines to reduce the risk of infection after surgery and helps the incision line to heal quickly and safely. Incision Sets are Compatible with Confort Npwt C300, C300 TS and C500 Therapy units. It is used with Continuous and Intermittent Therapy Modes. Two different size options are offered.



CODE	DESCRIPTION	SIZE
8680014006681	Incision Set Large	10*30
8680014004700	Incision Set X-Large	10*40

## White Foam Sets

Vacuum Assisted Wound Closure Sets White are compatible with Confort Npwt C300, C300 TS and C500 Therapy units. It is used with Continuous and Intermittent Therapy Modes. It is produced in 2 different sizes according to the size of the foam.



CODE	DESCRIPTION
8680014004809	Vacuum Assisted Wound Closure Set White Small (Hydrophobic)
8680014004786	Vacuum Assisted Wound Closure Set White Large (Hydrophobic)

## ACCESSORIES

### Y Connector

It helps two different wounds of the patient to be treated by a single Confort Npwt Therapy unit.



### Connector Cartridge For Instillation Treatment

Instillation therapy mode can be managed with Connector Cartridge for Instillation Treatment. It helps to transfer the solution (liquid) to the wound in sterile conditions.



CODE	DESCRIPTION
8680014004885	Vacuum Assisted Closure System Y Connector

CODE	DESCRIPTION
8680014006872	Vacuum Assisted Connector Cartridge For Instillation Treatment


# Confort

PRODUCTION TECHNOLOGY  
INNOVATION  
HEALTHY COMFORTABLE  
LIFE STANDARDS



## PRODUCT LIST

BARCODE	REF.NO	DESCRIPTION
8680014005868	CNFTU01	Confort Npwt Therapy Unit C300
8680014007503	CNFTU02	Confort Npwt Therapy Unit C500
8680014007312	CNFTU03	Confort Npwt Therapy Unit C300 TS
8680014004779	FD90001	Vacuum Assisted Wound Closure Set Small (Hydrophobic)
8680014004755	FD90013	Vacuum Assisted Wound Closure Set Medium (Hydrophobic)
8680014004687	FD90002	Vacuum Assisted Wound Closure Set Large (Hydrophobic)
8680014004724	FD90010	Vacuum Assisted Wound Closure Set with Silver Small
8680014004717	FD90014	Vacuum Assisted Wound Closure Set with Silver Medium
8680014006957	CFDSLED	Vacuum Assisted Wound Closure Set with Silver Large
8680014006841	CISS001	Vacuum Assisted Wound Closure Set Small Spiral For Instillation Treatment
8680014006858	CISS002	Vacuum Assisted Wound Closure Set Medium Spiral For Instillation Treatment
8680014006865	CISS003	Vacuum Assisted Wound Closure Set Spiral For Instillation Treatment
8680014003735	FDF0001	Canister 1000ml (Stop&Filter)
8680014006872	CNNCTRYI	Vacuum Assisted Connector Cartridge For Instillation Treatment
8680014004885	CNNCTRY	Vacuum Assisted Closure System Y Connector
8680014006933	FDSFADS	Vacuum Assisted Abdomen Management System (Extra Drape)
8680014006681	FDF0006	Incision Closure Set Large
8680014004700	FD90003	Incision Closure Set Extra Large
8680014004809	FD90006	Vacuum Assisted Wound Closure Set White Small (Hydrophobic)
8680014004786	FD90007	Vacuum Assisted Wound Closure Set White Large (Hydrophobic)
8680014007350	IMTC024	Canister Set-S Stop&Filter 500ML
8680014007336	IMTC022	Canister Set- S Stop&Filter 1000ML

 Konfort Özel Sağlık Hizmetleri Tıbbi Malzemeler San. Ve Tic. Ltd. Şti  
Address: Organize Sanayi Bölgesi 9. cadde No: 32 26110 Eskişehir / TURKEY  
Phone : +90 222 2360327 Fax : +90 222 2360323  
E-posta: info@confortnpwt.com - export@confortnpwt.com

[www.confortnpwt.com](http://www.confortnpwt.com)



1783- 2292

 [confort.tr](https://www.instagram.com/confort.tr)

 [konfortozel](https://www.linkedin.com/company/konfortozel)

 [CONFORT Npwt & Ostomy](https://www.youtube.com/channel/UC...)